
S-OPTIPRO (SLP-O) Eye Protective Goggles

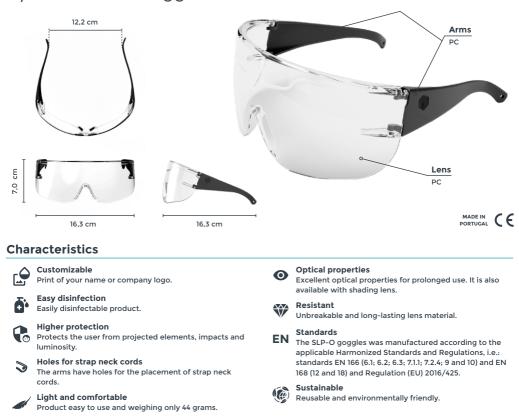



There are no known contraindications to the use of SLP-O reference Goggles, however, materials that may come into contact with the wearer's skin may cause allergic reactions in susceptible individuals.

Storage and Cleaning

Adapted for man and woman.

One size

It can be stored and transported in its original packaging at room temperature (0°C to 30°C). Can be stored in dark environments for up to 3 years. **Do not store in places where directly exposed to light and heat.** Clean with warm water, soap and a soft cloth or with a non-abrasive disinfectant solution. **Do not use solvents or abrasives**.

Marking

The SLP-O reference goggles have a marking with the following meaning:

Marking on the eyepiece: 3 PLASTIMAGO meaning that these are Safety Classes with good colour recognition, manufactured by PLASTIMAGO Lda.

<u>Marking on the support frame:</u> **PLASTIMAGO EN166 3** meaning that they are Safety Glasses with protection against splashes of liquids, manufactured by PLASTIMAGO Lda.

Validity

If the glasses are pitted, scratched or damaged, they must be replaced with an original equivalent.

Declaration of conformity available at em www.s-lifepro.pt

PLASTIMAGO Lda. | Estrada de Leiria, Nº 208 Embra 2430-091 | Marinha Grande, Portugal

